


RELIEF FOR PAIN SYMPTOMS


4 ELECTRODE PLACEMENT GUIDE

The placement of electrodes can be one of the most important parameters in achieving success with therapy. Apply electrodes to the exact site indicated by your physician or healthcare professional. The skin at the application site must be clean and dry. Press electrodes firmly to the skin, making good contact between the skin and the electrodes.


Never remove the self-adhesive electrodes from the skin while the device is turned on.

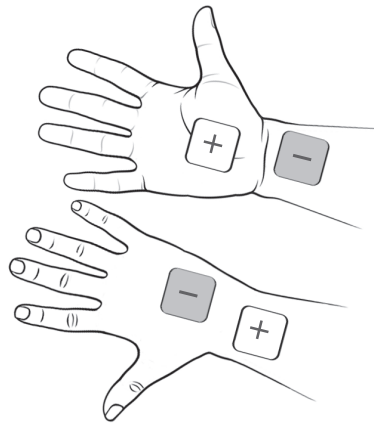


CARPAL TUNNEL

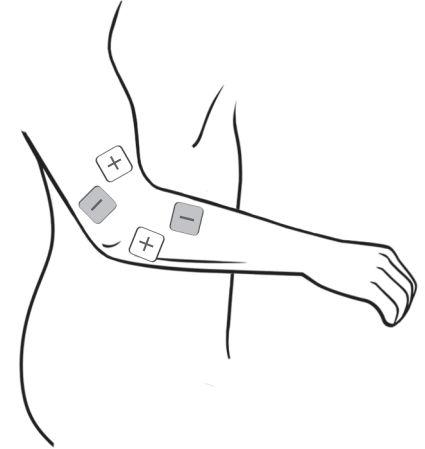


ELBOW PAIN

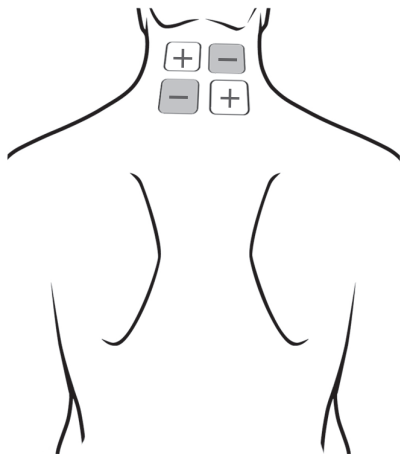


CERVICAL PAIN

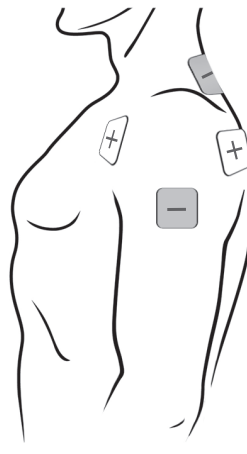


SHOULDER PAIN

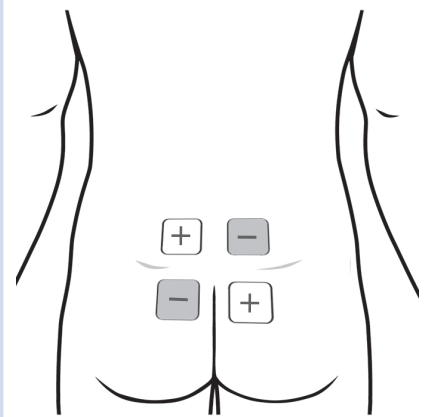


LUMBAR PAIN

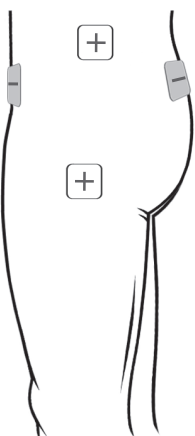

HIP PAIN

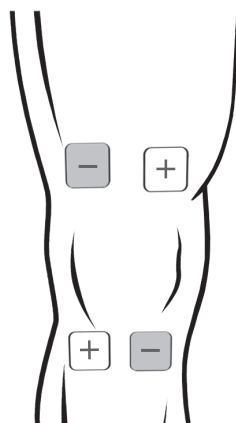

KNEE PAIN

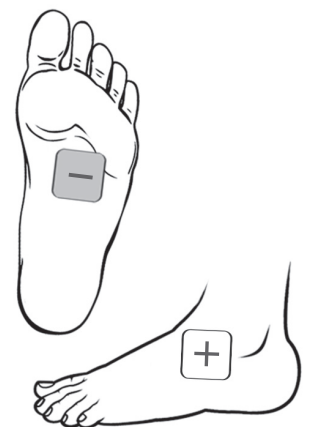


FOOT PAIN

RELIEF FOR PAIN SYMPTOMS


2 ELECTRODE PLACEMENT GUIDE

The placement of electrodes can be one of the most important parameters in achieving success with therapy. Apply electrodes to the exact site indicated by your physician or healthcare professional. The skin at the application site must be clean and dry. Press electrodes firmly to the skin, making good contact between the skin and the electrodes.


Never remove the self-adhesive electrodes from the skin while the device is turned on.

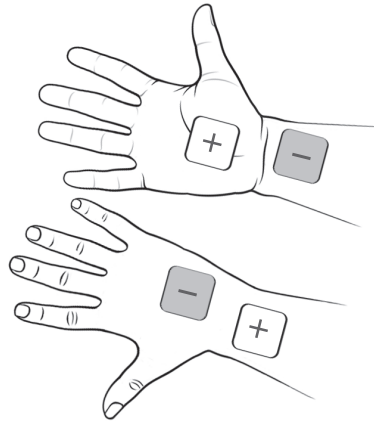


CARPAL TUNNEL

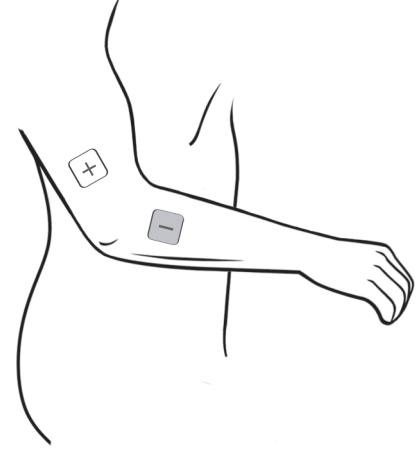


ELBOW PAIN

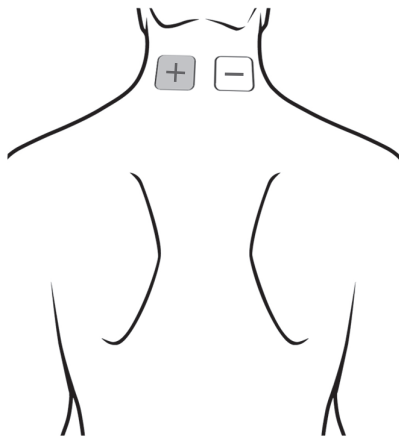


CERVICAL PAIN

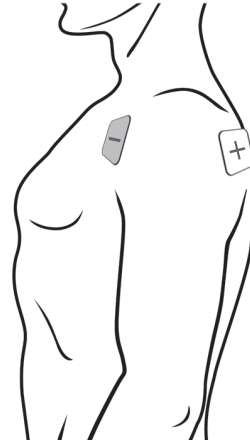


SHOULDER PAIN

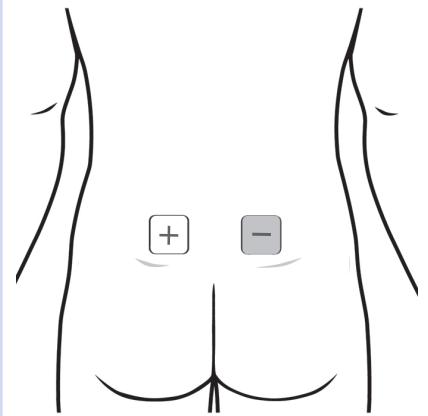


LUMBAR PAIN

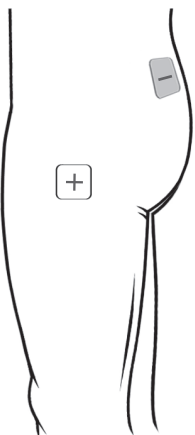

HIP PAIN

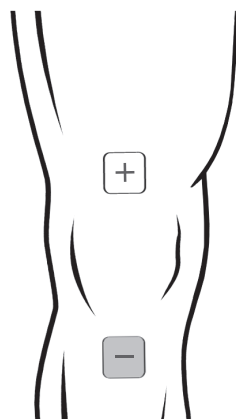

KNEE PAIN

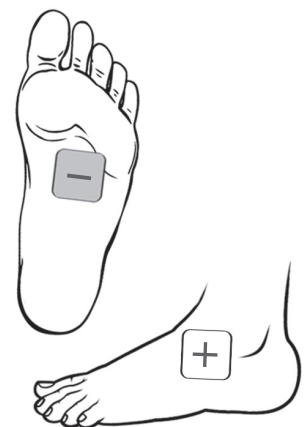

FOOT PAIN