



FRONT

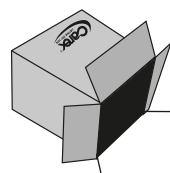

**B150
B154**




INSTALLATION

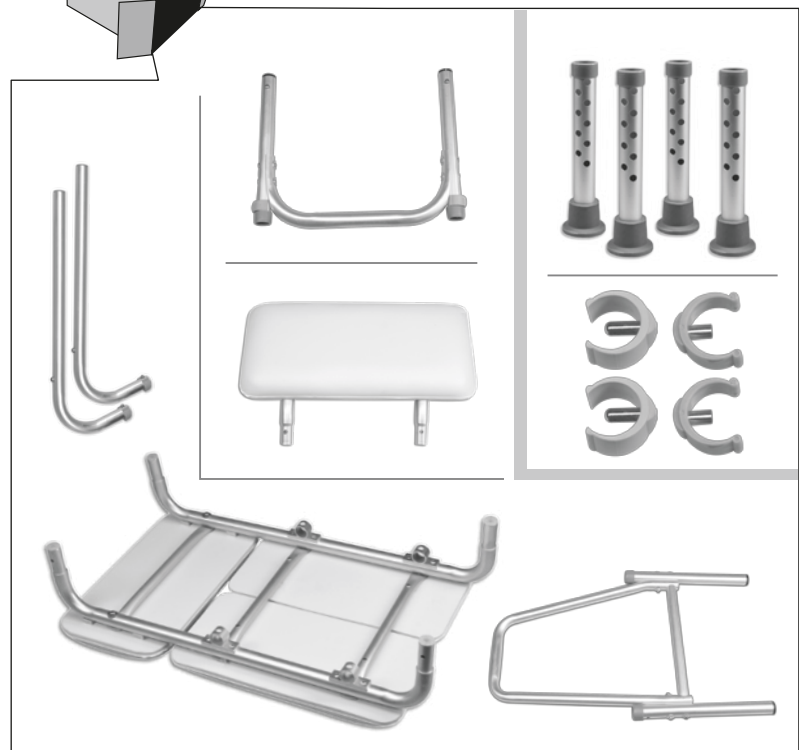

NOTE: BENCH CAN BE CONFIGURED IN SEVERAL DIFFERENT WAYS.

Best if this installation is done near the bathtub where the transfer bench will be used so seat back and handle bar are in the right place.

1

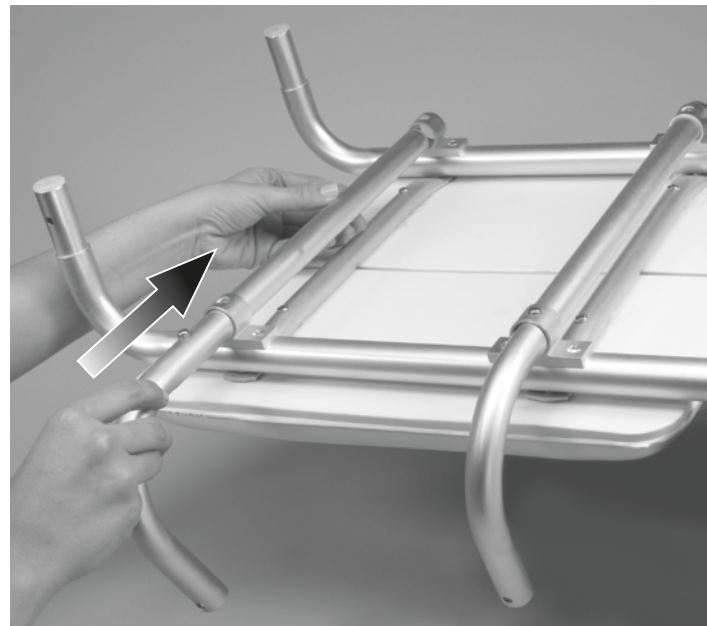

Attach elbow to arm as shown using bolt with washer and wing nut.
Repeat with other side.

2



Attach elbow to arm as shown using bolt with washer and wing nut.
Repeat with other side.


BACK

3

Slide leg attachments onto bottom of arm rests
Make sure pin has clicked into holes and that both legs are on the same setting so toilet safety frame will be level.



4

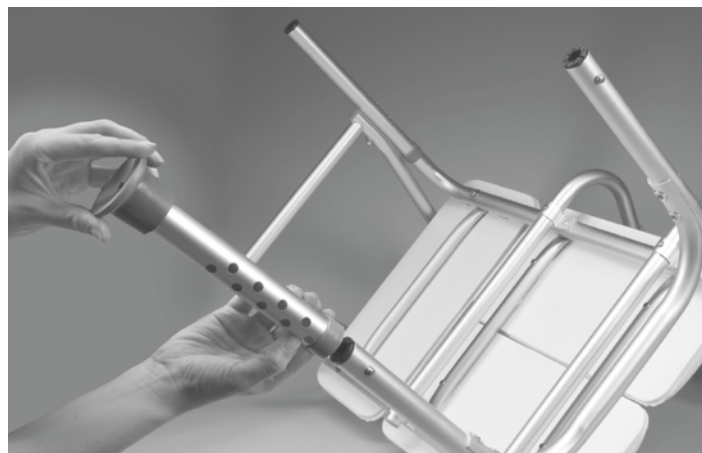

Carex Health Brands
PO Box 2526 • Sioux Falls, SD 57101-2526
Phone: 800-328-2935 • Fax: 888-616-4297
www.carex.com



All Rights Reserved
© 2009 Carex Health Brands
40-B150-00_01