Thera-Med Compression Back Wrap

CAUTION: Follow these instructions carefully. Misuse can cause burns. Overheating the clay pad in the microwave can damage or destroy product. If any swelling of the pad is observed while heating, turn off the microwave immediately and wait for swelling to go down before opening microwave door.

Before heating, the pad should be at room temperature. If you store the pad in the freezer, follow instructions for HEATING FROM FREEZER below.

Place pad in the center of the microwave and heat at HIGH power setting for 45 seconds.

If microwave does not have a turntable, reduce the initial heating time by 10 seconds and rotate the pad 90 degrees halfway through heating cycle.

Do not leave unattended while heating.

If higher heat is desired, continue microwaving in 15 second intervals, not to exceed 1 minute and 15 seconds total heating time.

Allow pad to rest in microwave for 1 minute. Test pad prior to use. If it feels too hot, allow it to cool until it is comfortable to touch.

Always place the clay pad inside the pocket of the included compression wrap after heating. Clay pad is not intended for use without a cover.

Continuous low heat in one area can burn you. Continually shift the position of the clay pad while in use.

REHEATING: If the pad is to be reheated after 30 minutes of use, follow above instructions, but reduce microwave time by half.

HEATING FROM FREEZER: Place the pad in the microwave on defrost setting or 30% Power Level for 60 seconds. Allow to stand for 5 minutes, and then follow normal heating instructions.

FOR COLD THERAPY: Place the pad in your freezer for at least one hour. Frozen pad will remain flexible and conform to your body. Comfort level could last up to one hour.

Apply the pad to the area you wish to cool for 20 minute intervals, allowing for 20 minutes off before reapplying to treatment area. Should there be discomfort, discontinue use and consult your physician.

DO NOT use this product on persons with nerve damage, circulatory problems, sensitive skin, or with damaged or broken skin; or on persons unable to remove this product such as infants, children, and some older persons.

Manufactured by Carex Health Brands, PO Box 2526 Sioux Falls, SD 57101 USA
If you have any questions or comments please call Carex Health Brands at
(800) 526-8051 Cloth cover made in China

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